



**ClearCheck SLP - Sealant and Liquid Polish**  
**DIRECTIONS FOR USE AS SEALANT**

ClearCheck SLP is used to seal tooth surfaces, temporary restorations, composite margins, orthodontic applications, and the pits and fissures of teeth. ClearCheck SLP contains no solvents, and will cure with all dental lights. ClearCheck SLP releases fluoride and is comprised of hydrophilic resins. These provide optimum properties for sealing and excellent tooth wetting. ClearCheck SLP has an extremely high compressive strength. This strength combined with excellent sealing and great esthetics make an excellent choice for sealing of teeth. ClearCheck SLP is approved for use with caries detecting devices.

**DIRECTIONS FOR SEALANT APPLICATION**

1. **Clean:** Clean the surface to be sealed with an oil-free prophylaxis paste. Rinse and dry.
2. **Isolate:** It is important that tooth surfaces to be sealed be kept free of contamination throughout the procedure.
3. **Etch:** Apply etchant for 15 seconds to the tooth surface to be sealed. Thoroughly wash off etchant. Lightly dry etched tooth surface. Do not desiccate. No bonding agents are needed.
4. **Apply ClearCheck SLP:** Place applicator tip on syringe and start flow of ClearCheck SLP. Use a new applicator tip for each patient. Apply material to surface, lesions, cervical erosion, or pits and fissures, as indicated.
5. **Light Cure:** Light cure for 20 seconds.
6. **Inspect the treated surfaces:** If more material is needed and the surface has not been contaminated, simply apply new material and cure for 20 seconds. If the surface has been contaminated, re-etch for 10 seconds, wash, lightly dry and place additional material. Cure for 20 seconds.
7. **Check occlusion:** Adjust as required.

**WARNING:**

- For professional use by dental personnel only.
- ClearCheck SLP and etchant may cause irritation to eyes and skin on contact. Take all necessary precautions to avoid contact with eyes and soft tissue. If materials contact eyes, rinse with water. If materials contact skin, wash with soap and water. Seek medical attention if irritation persists.
- Wear protective eyewear and gloves. Wash hands after use. Emergency eye wash fountain should be available.

**STORAGE**

- Store tightly capped in original container at cool room temperature. Avoid direct sunlight and extremes of temperature.
- Shelf life of unopened product: 2 years from date of manufacture. Re-cap immediately after use.
- Designed to be used at room temperature of 20-24° C or 68-75° F.



**ClearCheck SLP- Sealant and Liquid Polish**  
**DIRECTIONS FOR USE AS LIQUID POLISH**

ClearCheck SLP is used to seal tooth surfaces, temporary restorations, composite margins, orthodontic applications, and the pits and fissures of teeth. ClearCheck SLP contains no solvents, and will cure with all dental lights. ClearCheck SLP releases fluoride and is comprised of hydrophilic resins. These provide optimum properties for sealing and excellent tooth wetting. ClearCheck SLP has an extremely high compressive strength. This strength combined with excellent sealing and great esthetics make an excellent choice for sealing of teeth. ClearCheck SLP is approved for use with caries detecting devices.

**DIRECTIONS FOR LIQUID POLISH - SEALER APPLICATION**

1. **Clean:** Clean the surface to be sealed with an oil-free prophylaxis paste or finish back a temporary restoration or composite restoration to remove any un-cured, oily surface material that may be present.
2. **Dry:** The surface to be bonded, composite or temporary, should be dry. Temporary restorations do not need to be etched and bonding agents are not required.
3. **Apply ClearCheck SLP:** Place applicator tip on syringe and start flow of ClearCheck SLP. Apply material to surface in a thin layer. Air disperse, if desired.
4. **Light Cure:** Light cure all material for 20 seconds, each surface.
5. **Examine Material:** Examine material for complete coverage. Add additional material or finish any excess, as desired.

**WARNING:**

- For professional use by dental personnel only.
- ClearCheck SLP and etchant may cause irritation to eyes and skin on contact. Take all necessary precautions to avoid contact with eyes and soft tissue. If materials contact eyes, rinse with water. If materials contact skin, wash with soap and water. Seek medical attention if irritation persists.
- Wear protective eyewear and gloves. Wash hands after use. Emergency eye wash fountain should be available.

**STORAGE**

- Store tightly capped in original container at cool room temperature. Avoid direct sunlight and extremes of temperature.
- Shelf life of unopened product: 2 years from date of manufacture. Re-cap immediately after use.
- Designed to be used at room temperature of 20-24° C or 68-75° F.

