

What's Hot and What's Getting Hotter

BY HOWARD S. GLAZER, DDS, FAGD

MATERIAL

ClearCheck® SLP
Shofu Dental Corporation
1225 Stone Drive
San Marcos, CA 92078
800.827.4638
www.shofu.com

Everybody loves shiny objects and dentists are no exception; we want the surfaces of our composite restorations and provisionals to be shiny and smooth. Shofu's ClearCheck® SLP can help—better yet, it offers two kinds of help. First, it can be used as a pit and fissure sealant and used around orthodontic brackets. Second, it can be utilized as a liquid polish. Thus, ClearCheck SLP is a two-for-one special: a liquid polish and a sealant. In either instance, you can easily dispense the syring-

able, fluoride-releasing liquid and photocure the area in only 20 seconds. Any type of light source (including plasma arc, halogen, or LED) can be used with equal effectiveness. ClearCheck SLP has a high compressive strength; best of all, it can be used with the newest high-tech caries detecting devices. ClearCheck SLP is composed of hydrophilic resins, has no solvent, and is free of Bisphenol-A; as a result, it's great as a sealant. Simply isolate and dry the surfaces to be sealed, etch for 15 seconds, wash off the etchant, apply ClearCheck SLP,

and photocure for 20 seconds. To polish a provisional restoration, simply dry the surface(s) (no etch is required), apply ClearCheck SLP, and photocure for 20 seconds. ClearCheck SLP is a great two-in-one product that shines—forgive the pun—in daily practice.



Reprinted with permission from AGD Impact, March 2009. On the web at www.agd.org.
© Academy of General Dentistry. All rights reserved. Foster Printing Service: 866-879-9144, www.marketingreprints.com.



MANUFACTURER

SHOFU INC. 11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0983, Japan
SHOFU INC. Singapore Branch 10 Science Park Road #03-12, The Alpha, Science Park II, Singapore 117684
SHOFU DENTAL CORPORATION 1225 Stone Drive, San Marcos, CA 92078-4059, USA
SHOFU DENTAL PRODUCTS LTD. Duke's Factory, Chiddingstone Causeway, Tonbridge, Kent, TN11 8 JU, UK
SHOFU DENTAL SUPPLIES (Shanghai) CO. LTD. No.645 Jiye Road, Sheshan Industry Park, Songjiang 201602, Shanghai, China
EU REPRESENTATIVE: SHOFU DENTAL GmbH Am Brüll 17, 40878 Ratingen, Germany