

# MonoCem<sup>®</sup>

SELF-ADHESIVE RESIN CEMENT



Twist to  
remove cap




Push here to bleed tube<sup>†</sup>  
before putting on mixing tip.  
Express 1-2mm onto pad  
<sup>†</sup>First use only



**1. Prepare Tooth**  
rinse and leave moist



**2. Restoration Preparation**  
place MonoCem directly into crown  
discard mixing tip and recap syringe

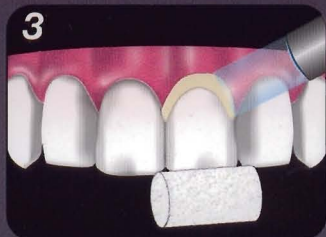


**3. Restoration Seating**  
place crown onto preparation  
FLASH cure excess 1-2 seconds



**4. Remove Excess Cement**  
use a suitable instrument and  
gently remove excess



**5. PFM Non-Disturb Phase**  
maintain positive pressure for 2-1/2 minutes  
then light cure margins for 20-40\* seconds  
with complete auto cure in 3-1/2 minutes  
REMOVE any additional cement

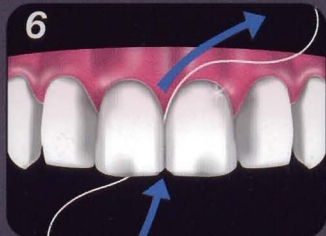

**6. Clear and Clean Contacts**  
floss up/down and pull  
through to clean margins

\*Light cure times vary. All ceramic, crown and inlay/onlay cementation light cure for 20-30 seconds. Post and core cementation light cure for 40 seconds. Refer to MONOCHEM DIRECTIONS FOR USE for more detailed instructions.

CEMENTS

