

Placing Attachments for Aligners using Beautifil® II



Visit www.shofu.com or call 800-827-4638



1. Uncut enamel requires acid etch - Complete phosphoric acid etch process. Apply **BeautiBond®** to etched surface and leave on for 10 seconds.



2. Air Dry – Blow air gently without scattering for 3 seconds then strongly for 2-3 seconds more.



3. Light cure **BeautiBond** – Halogen light for 10 seconds, LED light for 5 seconds.




4. Thoroughly dry aligner and fill **Beautifil® II** into attachment well slightly overfilling.



5. Fully seat loaded template onto the teeth.



6. Apply light pressure around each attachment to ensure full adaptation.

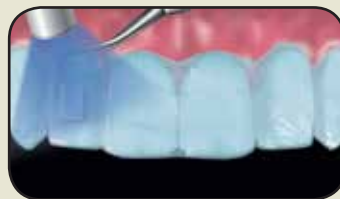

7. Firmly hold down **Beautifil II** with an instrument and light cure – 20 seconds for halogen and 10 seconds for LED.



8. Remove attachment aligner. Remove all flash and bonding resin between teeth. Floss interproximal areas. Polish all attachments and place the first aligner.

Placing Attachments for Aligners using Beautiful Flow Plus®



Visit www.shofu.com or call 800-827-4638



1. Uncut enamel requires acid etch - Complete phosphoric acid etch process. Apply **BeautiBond**® to etched surface and leave on for 10 seconds.



2. Air Dry – Blow air gently without scattering for 3 seconds then strongly for 2-3 seconds more.



3. Light cure **BeautiBond** – Halogen light for 10 seconds, LED light for 5 seconds.



4. Thoroughly dry aligner and fill **Beautiful Flow Plus**® into attachment well slightly overfilling.



5. Fully seat loaded template onto the teeth.



6. Apply light pressure around each attachment to ensure full adaptation.



7. Firmly hold down **Beautiful Flow Plus** with an instrument and light cure – 20 seconds for halogen and 10 seconds for LED.



8. Remove attachment aligner. Remove all flash and bonding resin between teeth. Floss interproximal areas. Polish all attachments and place the first aligner.