















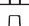



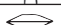






Instructions for Use

An all-inclusive BurButler CADMaster CAD/CAM Kit is indicated for contouring, finishing, polishing and fine polishing of all CAD/CAM materials, including feldspathic porcelain, lithium disilicate, leucite-reinforced ceramics, Zirconia and zirconia-reinforced lithium silicate, hybrid materials and CAD/CAM composites. This universal kit incorporates Shofu's Dura-Green Stones, Dura-White Stones, Dura-Green DIA Stones, CeraMaster, CeraMaster Coarse, and ZilMaster abrasives which are available in different shanks (HP, FG, CA), head diameters, shapes, and working lengths.

NOTE ON USE: All instruments are reusable and can be reprocessed using an ultrasonic cleaner or a washer disinfectant followed by sterilization in a dental autoclave.

- Impregnated with silicon carbide, **Dura-Green Stones** are used to contour and finish composites, porcelain, precious and non-precious alloys, and amalgam.
- Composed of micrograined aluminum oxide, **Dura-White Stones** achieve fine finishing on enamel, porcelain, composites and glass ionomer restorations.
- **Dura-Green DIA Stones** are vitrified bonded diamond abrasives indicated for contouring and finishing of Zirconia, porcelain and ceramic materials including lithium disilicate.
- A two-step polishing system, **CeraMaster** and **CeraMaster Coarse** together create a satin finish on enamel, porcelain, and Zirconia.
- **ZilMaster** is a three-step system for contouring, finishing, pre-polishing and polishing of Zirconia and lithium disilicate.

Row #	No. of pcs	PN	Image	Abrasive	Shank	Purpose		Recommended RPM	Max RPM
1	1	128C		CeraMaster Coarse, WH6	HP	Polishing	porcelain, ceramics, Zirconia	10,000-15,000	20,000
	1	0048		Dura-Green Stone, WH6	HP	Contouring, Finishing	porcelain, ceramics, Zirconia, CAD/CAM and lab composites; amalgam, precious/non-precious alloys	5,000-20,000	30,000
	1	0010		Dura-Green Stone, FL2	HP				
	1	0060		Dura-Green Stone, FL2	CA				
	1	0155		Dura-Green DIA Stone, TC4	HP	Contouring, Finishing	porcelain, ceramics, Zirconia		
2	1	0128		CeraMaster, WH6	HP	Polishing	porcelain, ceramics, Zirconia	10,000-15,000	20,000
	1	0022		Dura-Green Stone, IC9	HP	Contouring, Finishing	porcelain, ceramics, Zirconia, CAD/CAM and lab composites; amalgam, precious/non-precious alloys	5,000-20,000	30,000
	1	0203		Dura-White Stone, FL2	HP	Finishing, Pre-polishing	porcelain, ceramics, CAD/CAM and lab composites		
	1	0223		Dura-White Stone, FL2	CA				
	1	0154		Dura-Green DIA Stone, KN7	HP	Contouring, Finishing	porcelain, ceramics, Zirconia		

Row #	No. of pcs	PN	Image	Abrasive	Shank	Purpose		Recommended RPM	Max RPM
3	1	0126		CeraMaster, Bullet	HP	Polishing	porcelain, ceramics, Zirconia	10,000-15,000	20,000
	1	126C		CeraMaster Coarse, Bullet	HP	Pre-polishing			
	1	0124		CeraMaster, Mini-Pt.	CA	Polishing			
	1	124C		CeraMaster Coarse, Mini-Pt.	CA	Pre-polishing			
	1	0643		ZilMaster, Coarse, WH6	HP	Contouring, Finishing	Zirconia, lithium disilicate	5,000-15,000	20,000
4	1	0121		CeraMaster, Bullet	CA	Polishing	porcelain, ceramics, Zirconia	10,000-15,000	20,000
	1	121C		CeraMaster Coarse, Bullet	CA	Pre-polishing			
	1	0244		Dura-White Stone, FL2	FG	Finishing, Pre-polishing	porcelain, ceramics, CAD/CAM and lab composites	5,000-20,000	30,000
	1	0104		Dura-Green Stone, FL2	FG	Contouring, Finishing	porcelain, ceramics, Zirconia, CAD/CAM and lab composites; amalgam, precious/non-precious alloys		
	1	0649		ZilMaster, Medium, WH6	HP	Finishing, Pre-polishing	Zirconia, lithium disilicate	5,000-15,000	20,000
5	1	0123		CeraMaster, Cup	CA	Polishing	porcelain, ceramics, Zirconia	10,000-15,000	20,000
	1	123C		CeraMaster Coarse, Cup	CA	Pre-polishing			
	1	0136		CeraMaster, Mini-Pt.	FG	Polishing			
	1	0248		Dura-White Stone, RD2	FG	Finishing, Pre-polishing	porcelain, ceramics, CAD/CAM and lab composites	5,000-20,000	30,000
	1	0655		ZilMaster, Fine, WH6	HP	Polishing	Zirconia, lithium disilicate	5,000-15,000	20,000

Distributed by
Shofu Dental Corporation
1225 Stone Drive, San Marcos, CA, USA

www.shofu.com • (800) 827-4638