

BEAUTIFIL Flow Plus® X

BIOACTIVE INJECTABLE HYBRID RESTORATIVE
FOR ALL INDICATIONS

Visit www.shofu.com | Call 800.827.4638

SHOFU



WARNING:
HIGH GLOSS
DISTRACTION




F03
LOW FLOW



F00
ZERO FLOW

EFFORTLESS POLISHING

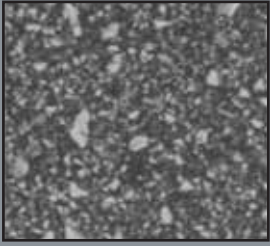

Micro-Hybrid filler particles



Nano-Hybrid filler particles

The combination and enhancement of two innovative technologies—SPR-G nano filler particles and stable surface glass ionomer—Beautiful Flow Plus X features improved mechanical properties, optimal polishability and long-lasting shine.



The surface gloss is clearly visible after light-curing where the remaining oxygen-inhibited layer was wiped off with a cotton ball. An effortless final polish will provide a sustainable high-gloss finish.

STRONG WITH SUPERB HANDLING

Beautiful Flow Plus X can be used as a base, liner and final restorative. With the compressive strength of 358 MPa and the flexural strength of 120 Mpa, Beautiful Flow Plus X can easily withstand high-stress loads on the occlusal surface and marginal ridge. Available in two viscosities, F00 (zero flow) and F03 (low flow), both deliver optimal anterior and posterior restorations. Excellent shape retention, F00, accurately restores occlusal morphology without slumping and F03 composite thoroughly wets the cavity walls and has superior adaptation.



Photos: Dr. Kenichiro Sadamitsu, Japan

NON-OOZING & SELF-LEVELING

For precise direct restorations, easily dispense the right amount of material with the newly developed thin-walled needle tips. The material neither sticks or oozes at the end of delivery.




BEAUTIFIL Flow Plus® X


BIOACTIVE INJECTABLE HYBRID RESTORATIVE FOR ALL INDICATIONS

Beautiful Flow Plus X is the latest bioactive injectable hybrid restorative in the Beautiful product line with a newly developed patented nano S-PRG filler. Beautiful Flow Plus X has all of the Giomer chemistry attributes, including fluoride release and recharge, now with improved handling and effortless polishing that maintains a long-lasting shine.

- Achieves more surface gloss in less time—effortless polishing
- Improved handling
- Stackable & sculptable; stays put
- Chameleon effect—aesthetically blends with the natural tooth
- High flexural strength
- Low wear resistance & shrinkage
- Indicated for anterior & posterior teeth, classes I-V

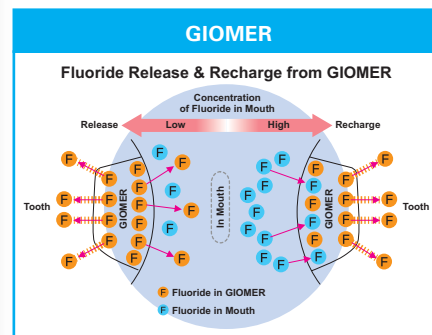

Giomer's continuous fluoride release and recharge capability contributes to long-term caries inhibition

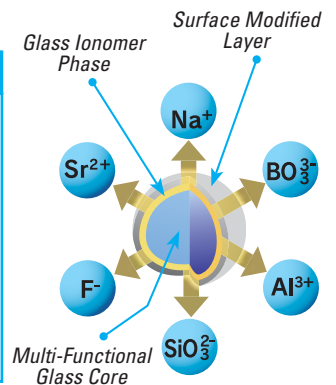

Six beneficial bioactive ions

BEAUTIFIL Flow Plus F00 (Zero Flow) 2.2g Syringe		BEAUTIFIL Flow Plus F03 (Low Flow) 2.2g Syringe	
Shade	Item No.	Shade	Item No.
A0.5	YN2300	A0.5	YN2320
A1	YN2301	A1	YN2321
A2	YN2302	A2	YN2322
A3	YN2303	A3	YN2323
A3.5	YN2304	A3.5	YN2324
A4	YN2305	A4	YN2325
A0.50	YN2310	A20	YN2332
A10	YN2311	A30	YN2333
A20	YN2312	B1	YN2326
B1	YN2306	B2	YN2327
B2	YN2307	C2	YN2328
C2	YN2308	D2	YN2329
D2	YN2309	INC	YN2334
INC	YN2314	BW	YN2335
BW	YN2315	MI	YN2336
		CV	YN2337



Shofu Dental Corporation
1225 Stone Drive
San Marcos, CA
92078-4059, USA