

# Niveous<sup>®</sup>

In-Office Whitening System



fast  
economical  
simple  
versatile  
effective



# Niveous®

## CASE STUDY



**One:**  
Use OneGloss® PS polishers to remove difficult surface stains and to prepare tooth surface



**Two:**  
Before Niveous.



**Three:**  
After Niveous.  
Photos courtesy of Jeffrey Hoos, DMD FAGD

- 25% stabilized hydrogen peroxide – rapid, yet long lasting results\*
- Conveniently packaged – unit dose, no mixing, quick and easy application
- Each 1mL ampule – delivers up to three patient applications per arch
- Improved Booster brushes are **three times stronger** – enhances the whitening effect
- Dual activated formula – use with or without any light source


Success in private practice is assured when the dentist finds the correct BALANCE. The BALANCE is between the ART, SCIENCE & BUSINESS of DENTISTRY. **Niveous** from Shofu allows the office to provide a beautiful bleaching result... ART. The product is well researched, with good documentation... SCIENCE. It can be delivered in a timely, cost effective manner, without the additional expense of a "light". In difficult economic times, everyone must cognizant of reducing overhead... BUSINESS.

**Jeffrey C Hoos, DMD FAGD**

Stratford, CT

Balancing the Art, Science & Business of Dentistry

Visit [www.shofu.com](http://www.shofu.com) or call **800.827.4638**

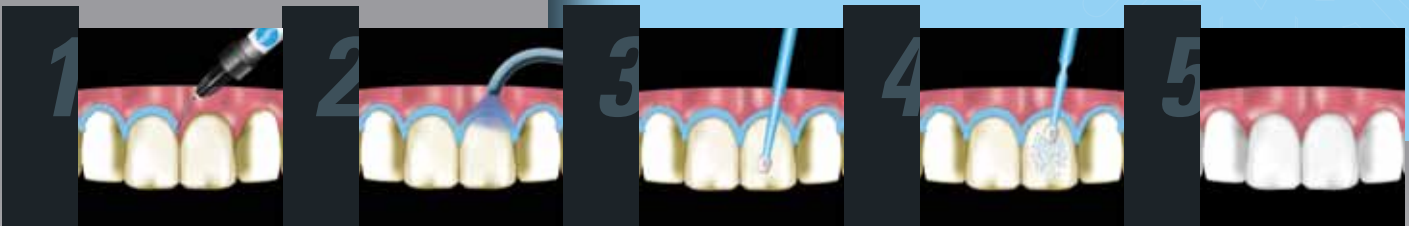

**One:** Syringe Liquid Dam onto gum tissue.

**Two:** Light cure material until polymerized entirely.

**Three:** Use Booster Brush to distribute gel from ampule.

**Four:** Activate the gel for 3 – 4 seconds per tooth.

**Five:** Beautiful results.



#### Order Information

- Part Number 1291
- (10) 1mL Niveous bleach gel ampules
- 3g syringe of Niveous Liquid Dam
- 30 Booster brushes
- Dappen dish
- Directions for Use

\*Niveous has the least reduction in brightness of 7 leading in-office whitening systems after one year, as reported by an internationally respected independent research institute. Study available upon request.



Shofu Dental Corporation  
1225 Stone Drive  
San Marcos, CA  
92078-4059, USA

SNF-2-0210

# In-Office Whitening System