

# CERARESin BOND


Porcelain & Resin Repair System

The logo consists of a blue circle with the word "SHOFU" in white, uppercase letters. The letters are slightly shadowed, giving a 3D effect. The logo is centered on a light beige horizontal band.

SHOFU



EASY PROCEDURE

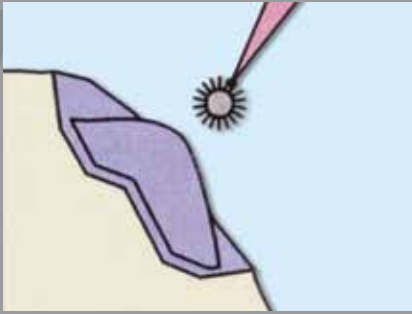

One: Apply Cerarasin Bond 1. Leave for 10 seconds.



Two: Apply Cerarasin Bond 2. Leave for 10 seconds.



Three: Light-cure for 20 seconds. Ready for composite.


Cerarasin Bond can be used to modify porcelain restorations quickly and efficiently with light-curing materials. Cerarasin Bond combines cold-curing resins with light-curing veneering material as well as different light-curing veneering composites. Also there are no limits to individualizing denture teeth with light-curing veneering material.

# CERARESIN BOND

## Porcelain & Resin Repair System

- Intra oral and lab repairs
- Bonds to porcelain, ceramic, resin, acrylic and metal\*
- Resistant to moisture and thermal strains
- Quick and easy to use
- Compatible with all resins and composites
- 3 year shelf life

Visit [www.shofu.com](http://www.shofu.com) or call **800.827.4638**

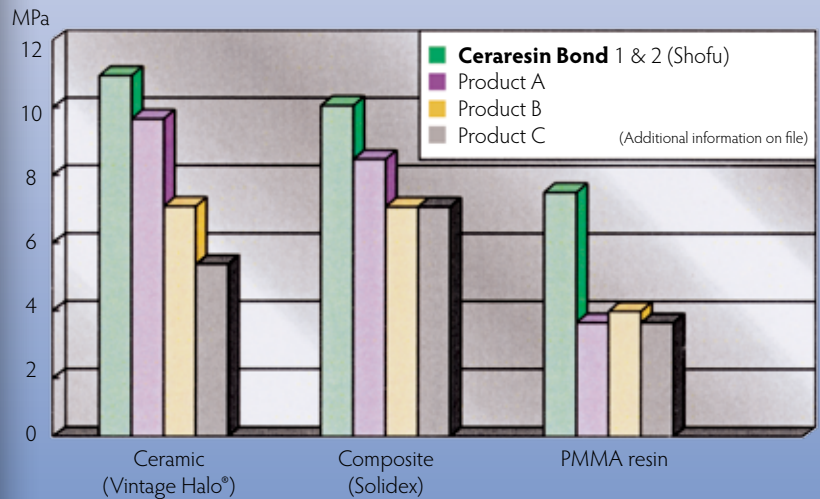

Bonding strength of composite and different materials with Cerarasin Bond as well as ceramics / resin-bonding systems of other manufacturers

\*Metal exposure requires ML Primer (metal/liquid primer) PN 1740



#### Order Information

- Part Number 1741
- 6mL CRB Bond 1
- 5mL CRB Bond 2
- 25 Microbrush Fine Yellow
- 25 Microbrush Fine Pink
- 5 Disposable Dish



Shofu Dental Corporation  
1225 Stone Drive  
San Marcos, CA  
92078-4059, USA

Porcelain & Resin  
Repair System

SN76-0413