



# INSTRUCTIONS FOR USE

## Amalgam & Metals Polishing Kit - FG

READ CAREFULLY BEFORE USE

0309B

### PRODUCT DESCRIPTION

This kit is designed for finishing, polishing and super-polishing amalgam, precious and non-precious alloys. The kit includes Brownies, Greenies and Supergreenies in (7) Mini-Pt., (5) Cup.

### INTENDED PURPOSE



Brownies: Pre-polishing  
Greenies: Polishing  
Supergreenies: Super-polishing

### PRECAUTIONS AND WARNINGS

- Never exceed the maximum permissible rotating speed.

Required Speed	5,000-30,000 rpm
----------------	------------------

- Operate at recommended pressure only.
- Insert the shank completely into the chuck of the hand piece, following the manufacturer's instructions.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Make sure that abrasive head is in good condition before use. Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt, exposure of shank due to repeated use.
- Use local dust extractor, protective mask, etc while contouring and finishing to avoid adverse effect of dust to the human body.
- Use protective equipment such as glasses while using.

SHAPE CATEGORY		MINI-PT.	CUP
SHAPE			
ISO#		030	065
Brownie			
FG (12 PK)	PN	0413	0411
Greenie			
FG (12 PK)	PN	0414	0412
Supergreenie (Yellow Band)			
FG (12 PK)	PN	414B	412A

- Avoid any use at acute angles or with excessive pressure to thin and long abrasives. They are fragile and may bend or fracture under excessive loads.
- This product is intended for use by dental professionals only.
- Use the points for indicated purpose only.

### **STORAGE AND STERILIZATION**

This kit can be sterilized in any steam or chemical autoclave. Store this product avoiding exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination. When using intraorally, sterilize this product before each use. For reuse after using, remove any contaminants immediately with washer disinfectant, then sterilize. Follow the instructions for use of washer disinfectant and sterilizer, respectively. Chlorine disinfectant must not be used to avoid corrosion of the instruments.

For the details about REPROCESSING procedures, visit [www.shofu.de](http://www.shofu.de).

### **DISPOSAL**

Dispose in accordance with relevant local and national regulation.

***CAUTION: US Federal law restricts this device  
to sale by or on the order of a dental professional***

DFU/0309B

#### **SHOFU DENTAL CORPORATION**

San Marcos, CA 92078 | [shofu.com](http://shofu.com) | 800.827.4638

#### **SHOFU INC.**

Kyoto 605-0983, Japan