

# BeautiBond®

THE SCIENCE BEHIND THE BOND®



1. Snap off the Cap



2. Insert a Brush to Dispense



3. Apply BeautiBond  
leave on for 10 seconds



4. Air-dry  
blow gently without scattering  
for 3 seconds – strongly  
for 2-3 seconds more



5. Light Cure  
halogen light for 10 seconds  
LED light for 5 seconds



6. Fill with Composite Material

RESTORATIVES



# Complementary Products

## Beautiful® II



Available in tips and syringes

Beautiful II is a nano-hybrid composite with fluoride release and recharge. A highly aesthetic dental material for all classes of restorations.

- Supported by 8 year clinical study by University of Florida
- Exceptional radiopacity
- Outstanding handling characteristics

## Beautiful Flow Plus®



Available in tips and syringes

Beautiful Flow Plus is the next step in the evolution of restorative materials, combining the delivery of a flowable and the strength, durability, and aesthetics equal to or better than leading hybrid composites.

- Indicated in classes I, II, III, IV & V
- Physical properties rival hybrids
- Stackable & sculptable; stays put
- Self-polishing "leveling-effect"



# RESTORATIVES



NOW IN BOTTLES!



For more information on these and many other great products visit [www.shofu.com](http://www.shofu.com) or call 800.827.4638

SN125-0815

Shofu Dental Corporation • San Marcos, CA