

BeautiBond®

Dual Adhesive, Self-Etch, Seven Generation Bonding System



BeautiBond
Dual Adhesive, Self-Etch,
Seventh Generation
Bonding System



**SERVE
PATIENTS
INCREASE
PRODUCTION
GO HOME
ON TIME!**

Shofu is dedicated to manufacturing the highest quality products utilizing our innovative technologies to enhance minimally invasive cosmetic dentistry.

shofu.com
800.827.4638
customer-service@shofu.com

HIGH STRENGTH

Thanks to **BeautiBond's** unique "dual adhesive" formula, dentists no longer have to buy multiple bonding systems. **BeautiBond's** chemistry features separate adhesives for both dentin and enamel so clinicians are guaranteed high bond strength regardless of the procedure or class of restoration.

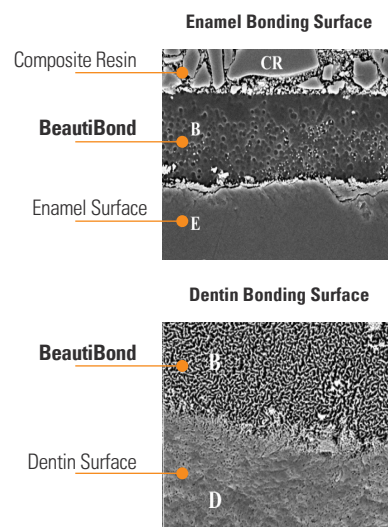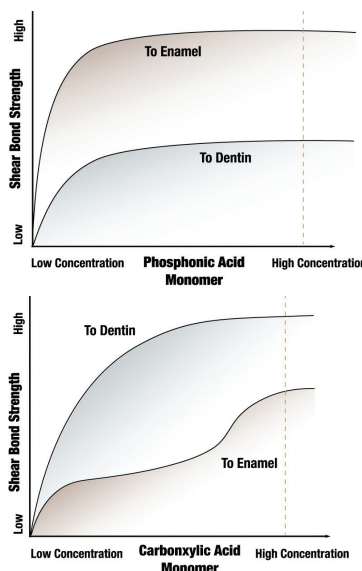

High enamel and dentin bond strength was achieved with **BeautiBond** despite less aggressive etching patterns¹

MINIMAL SENSITIVITY

Unlike etch-and-rinse systems which unavoidably expose delicate dentin to harsh acid or desiccation, **BeautiBond** seals and protects the dentin against these factors which can often cause sensitivity. Comparative 3-year clinical study showed 0% sensitivity.²



AESTHETICS

With an extremely low film thickness (less than 5µm), **BeautiBond** eliminates marginal stain lines caused by brands with thicker bonding layers; making **BeautiBond** the best choice when restorations must be picture perfect.




1. Dr. Camila Sabatini independent study SUNY at Buffalo, School of Dental Medicine Department of Restorative Dentistry, Buffalo, NY, USA. Presented at the International Dental Adhesives Meeting.

2. M.A. LATTA et al., J Dent Res 92 (Spec Iss A):176524, 2013 (www.dentalresearch.org). Xeno IV is a registered trademark of Dentsply.




FAST AND EASY TO USE


BeutiBond is so efficient that it will increase productivity. Apply, dry, and cure – nothing is easier!

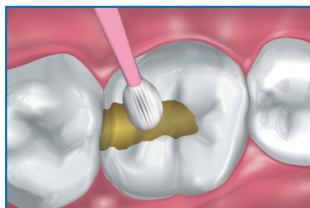

Apply **BeutiBond** into the prep and leave for 10 seconds.

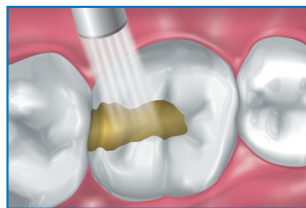

Gently air dry for about 3 seconds then blow with greater force.

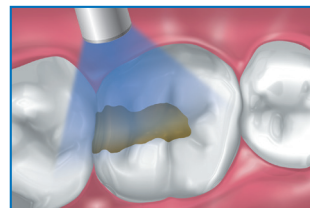

Light cure with a halogen (10 sec.) or LED (5 sec.) curing light.

PROVEN QUALITY

For over a decade, **BeutiBond** has been used with confidence by dentists all over the world due to its combination of strength, minimal sensitivity, aesthetics, and ease of use. In addition, it has been evaluated by numerous research facilities and institutions; always receiving among the highest ratings.



CONFIDENCE

For over 100 years, Shofu Dental's dedication to quality has made us one of the world's most respected dental products manufacturers. What can we do for you?

shofu.com
800.827.4638
customer-service@shofu.com



BeutiBond

- ☐ **BeutiBond Bottle**
(Item No. 1781)
• 6mL, 50 micro brushes

- ☐ **BeutiBond Unit Dose**
(Item No. 1782)
• (50) 0.1mL unit doses
• 50 micro brushes



Scan Here
Instructions
for Use