 **Manufacturer**  
SHOFU INC.  
11 Kamitakamatsu-cho, Fukuine,  
Higashiyama-ku, Kyoto 605-0983, Japan  
www.shofu.com

 0123

 **EC REP**  
SHOFU DENTAL GmbH  
Am Brühl 17, 40878 Ratingen, Germany

SHOFU DENTAL CORPORATION  
1225 Stone Drive, San Marcos,  
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.  
10 Science Park Road, #03-12,  
The Alpha, Science Park II, Singapore 117684

03:2019-03  
71406

## CeraMaster Finishing & Polishing Kit

**EN**

READ CAREFULLY BEFORE USE

This assortment is recommended for systematic finishing and polishing of ceramic restorations. It includes three Dura-Green Stones, three Dura-White Stones, three CeraMaster Coarse Polishers (blue), and three CeraMaster Polishers (white and blue bands). First, use Dura-Green or Dura-White Stones for grinding and contouring the ceramic restoration. Then use CeraMaster Coarse for finishing and polishing and CeraMaster for super polishing. The CeraMaster, which are filled with a high density of pure diamond particles, are durable and designed for quick finishing and polishing of ceramic restorations to provide a smooth surface.

### PRECAUTIONS

- Never exceed the maximum permissible rotation speeds described in Table A.
- Read all instructions on the back side carefully before use.

### STERILIZATION

Perform autoclave under the conditions described in Table B.

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.**

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## (CAUTIONS)

- Never exceed the maximum permissible rotation speeds.
- Insert the shank completely into the chuck of the handpiece.
- Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Avoid any use at acute angles or with excessive pressure.
- Use local dust extractor and protective equipment while using.
- Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than the intended use.

## (STERILIZATION)

- Perform autoclave for 3 min at 134 °C (273 °F) or 30 min at 121 °C (250 °F) for sterilization.
- When using intraorally, sterilize before each use.
- For reuse after using intraorally, remove any contaminants immediately with cleaning liquid or disinfectant, then sterilize.
- Dura-Green Stones and Dura-White Stones: Do not use chlorine disinfectant liquid (sodium hypochlorite) and benzalkonium chloride solution. Do not perform plasma sterilization.
- CeraMaster: Do not immerse into chlorine disinfectant for a prolonged time to avoid corrosion. Do not perform dry heat sterilization.
- Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.

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