

	UNI-XS Dentalcolor XS - Kulzer	Liculite/de Trey Polylux HDS 400/Dreve	Labolight LV II + LV III GC
LIGHT CURE			
Pre-Opaque	1.5 minutes	3 minutes	2 minutes
Opaque	3 minutes	5 minutes	3 minutes
Composite (pre-light-cure)	1.5 minutes	3 minutes	2 minutes
Flowable composite (pre-l-c)	1.5 minutes	3 minutes	2 minutes
Pontic	3 minutes	5 minutes	3 minutes
Final	3 minutes	10 minutes	5 minutes