



Composite

POLISHING PASTE

One 5 gram syringe

PN 0551

Directions on Use

CompoSite Polishing Paste is specifically formulated to polish adjusted hybrid or microfill composites to a brilliant shine. Water soluble CompoSite Paste can be used with equal effectiveness on both anterior and posterior restorations. However, the quality of the polish is heavily dependent upon how well the restoration is finished prior to polishing. For optimal results, use the surface preparation instruments described below.

Surface Preparation:

Ideal surface preparation can be accomplished by using Shofu's Super-Snap® Disks or CompoSite and CompoSite Fine silicone rubber points.

Complete the recommended finishing/polishing sequence for each product, carefully observing the correct technique.

Polishing Paste Technique:

Apply a small amount of CompoSite Polishing Paste directly to the center of a moistened Super-Snap Buff Disk (PN L523) or Buff Mini-Disk (PN L524). To reduce splatter, spread the paste in a thin, even layer on the felt surface. Polish the adjusted surface using light buffing strokes at slow (5,000 - 10,000 RPM) speed. Keep the paste moist throughout the polishing procedure. When polishing is complete (approx. 30 seconds) remove any residual paste with water spray.

Note:

Always dispense CompoSite Polishing Paste carefully. Before dispensing paste, pull the syringe plunger back to relieve pressure, then push forward gently. If syringe tip is blocked, clear the tip by inserting a sterile pointed instrument.

For maximum shelf life store CompoSite Polishing Paste in a cool cabinet not subjected to direct sunlight or excessive heat.

SHOFU DENTAL CORPORATION

1225 Stone Drive, San Marcos, CA 92078-4059, USA (www.shofu.com)

SHOFU INC.

11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0983, Japan (www.shofu.co.jp)

SHOFU DENTAL TRADING (SHANGHAI) CO., LTD

No. 645 Jiye Road, Sheshan Industrial Park, Songjiang, Shanghai 201602, China (www.shofu.com.cn)

SHOFU INC. Singapore Branch

10 Science Park Road, #03-12 The Alpha, Science Park II, Singapore 117684 (www.shofu.com.sg)

SHOFU UK

Riverside House, River Lawn Road, Tonbridge, Kent, TN9 1EP, UK (www.shofu.co.uk)

SHOFU DENTAL GmbH

Am Brüll 17, 40878 Ratingen, Germany (www.shofu.de)

Printed in USA