


QUICK GUIDE

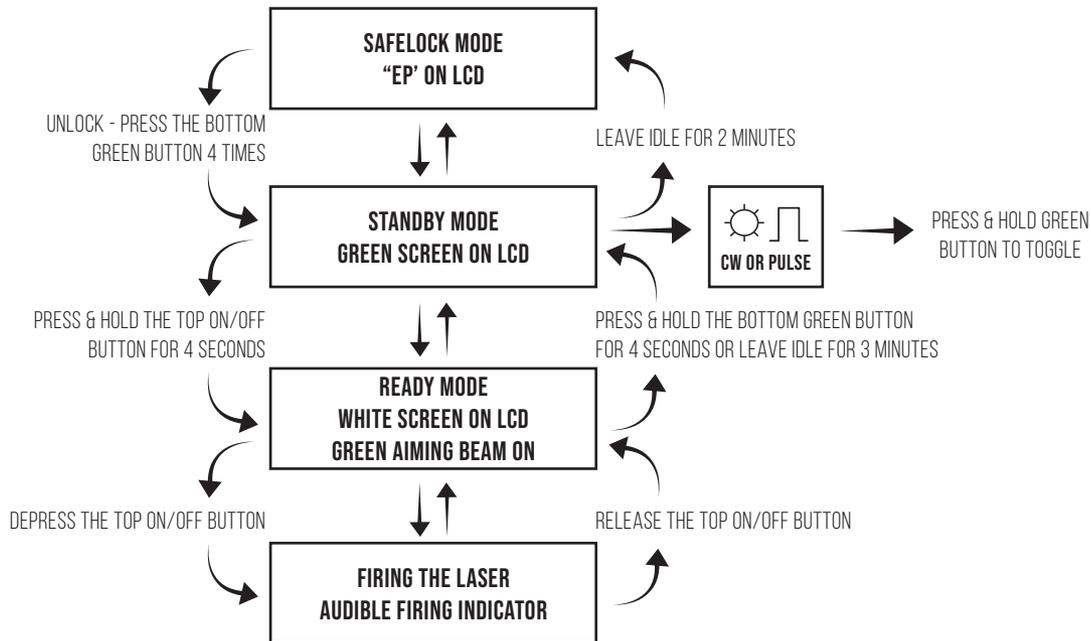

- KEEP COVERED WITH ORANGE CAP ON WHEN NOT IN USE
- AFTER REMOVING ORANGE CAP, PUT THE FIBER TIP ON IMMEDIATELY

- NON-CONTACT PROCEDURES NO TIP INITIATION NEEDED
- TO INITIATE THE FIBER TIP, TOUCH TIP TO BLACK PAPER
- THE FIBER TIPS CAN BE BENT UP TO 45°

Distributed by
Shofu Dental Corporation

(800) 827-4638
shofu.com



RECOMMENDED TREATMENT SETTINGS

To achieve the same rate of output in the Pulse Mode double the power from the CW Mode.

PROCEDURE	MODE	POWER	TECHNIQUE	TIP
Aphthous Ulcer/Canker Sores	CW	0.5	Non-contact	Non-initiated
Biopsy	CW	1.5	Contact	Initiated
Crown Lengthening	Pulse	2.0	Contact	Initiated
Excision/Incision	CW	1.5	Contact	Initiated
Expose Teeth	CW	1.0	Contact	Initiated
Fibroma	CW	1.5	Contact	Initiated
Frenectomy	CW	1.5	Contact	Initiated
Gingivectomy	CW	1.5	Contact	Initiated
Gingivitis	CW	0.5	Non-Contact	Non-Initiated
Gingivoplasty	CW	1.5	Contact	Initiated
Hemostasis	CW	1.0	Contact	Initiated
Herpetic Lesion	Pulse	1.5	Non-Contact	Non-Initiated
Hyperplasia	CW	1.0	Contact	Initiated
Implant Recovery	CW	1.5	Contact	Initiated
Periodontal Sulcular Debridement	Pulse	1.0	Non-Contact	Non-Initiated
Pulpotomy	Pulse	1.0	Non-Contact	Non-Initiated
Recontouring	CW	1.0	Contact	Initiated
Troughing	CW	1.0	Contact	Initiated

* Use minimum power above the threshold when cutting to avoid unnecessary lateral damage to the tissue for speedier cutting, less pain, and faster recovery for the patient. Loupes will help see the threshold power.