



## ENGLISH

DirectDia is a diamond impregnated polishing paste used both intra and extra orally for achieving a high luster polish on both the restoration and natural tooth surface.

### INDICATIONS

1. High luster polishing of restorative materials (porcelain, direct composite, lab composite, metals etc.)
2. Polishing and cleaning of natural teeth

### DIRECTIONS FOR USE

1. Remove cap from syringe and attach the delivery tip.
2. Apply a small quantity of paste to the Super-Snap Buff Disk (sold separately).
3. A dry surface is not required. Use on a saliva moist surface is acceptable.
4. Rinse thoroughly with water to remove all paste.

### CAUTIONS AND RECOMMENDATIONS

- Finish the surface with silicone polisher before using.
- Attach the delivery tip to syringe securely and tightly.
- Use with Super-Snap Buff Disks - standard or mini (Shofu PN L523 or L524).
- To prevent splatter and avoid excessive heat generation always use rotary instruments at speeds below 10,000 r.p.m.
- The quality of the product is not affected if liquid separation and pale color by liquid separation occurs. If the paste hardens it can be adjusted by adding water.
- Store in a dark place away from direct sunlight.
- Store at temperatures between 1°C and 30°C.
- Replace cap tightly after use to prevent drying.
- Wearing protective glasses is recommended for patient and operator.

### PACKAGE

DirectDia Paste: 1 syringe, 3g  
Tip with cap: 2 each