

FIT SA™

Self-Adhesive, Nano-Hybrid Flowable Composite Restorative with GIOMER Technology Inside — The Better Choice



FIT SA

Self-Adhesive,
Nano-Hybrid Flowable
Composite Restorative with
GIOMER Technology Inside
The Better Choice



**SERVE
PATIENTS
INCREASE
PRODUCTION
GO HOME
ON TIME!**

Shofu is dedicated to manufacturing the highest quality products utilizing our innovative technologies to enhance minimally invasive cosmetic dentistry.

shofu.com
800.827.4638
customer-service@shofu.com


SELF-ADHESIVE TECHNOLOGY

FIT SA has eliminated the need for the technique sensitive bonding procedure. This quantum leap in dental materials technology allows clinicians to use **FIT SA** as a liner, small Class I (PRR), Class III, Class V, and other non load-bearing restorations - without the need for a bonding agent! **FIT SA**'s unique, BPA-free chemistry provides a chemical bond as well as resin tag infiltration.

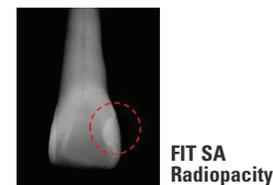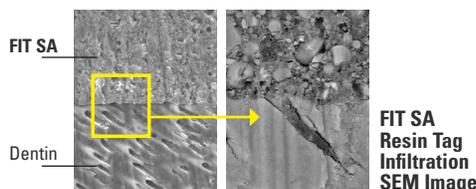

AESTHETICS

As a nano-hybrid, **FIT SA** offers superior strength and polishability. The unique filler structure combines the light transmission and diffusion properties of both dentin and enamel to blend well with surrounding dentition for picture perfect restorations.

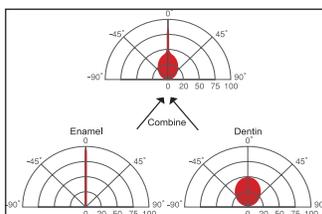

Light transmission and diffusion properties of FIT SA




Jack Griffin Jr., DMD

Mounika Falemban, DMD

GREATLY REDUCES SECONDARY DECAY (GIOMER)


Shofu's exclusive GIOMER technology gives **FIT SA** the ability to release and recharge fluoride and other beneficial ions for the life of the restoration. As a result of these bioactive properties, your patients receive healthful benefits unavailable in any competitor—anti-bacterial effect, acid neutralization, strengthen tooth structure and more! These benefits were validated by a clinical study at the University of Florida Dental School where no failures or decay were found in any of the restorations—even after eight years!*

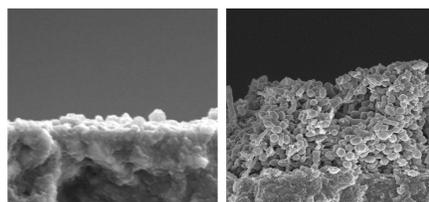

FIT SA with GIOMER
Minimal Bacteria

Competitive Composite
Advanced Bacterial
Colonization

Fluoride Release & Recharge from GIOMER

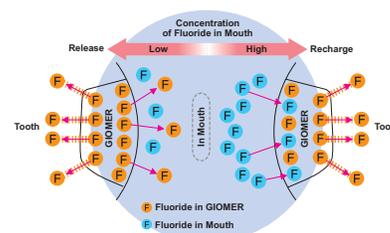


With the lack of a bonding layer, **FIT SA** allows for the free flow of healthful ions from the GIOMER restorative into the tooth structure.




FAST AND EASY TO USE


Liner (F10 Viscosity)

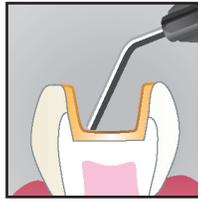

- Clean and gently air blow the prep. Then apply FIT SA.
- Spread in a thin layer ($\leq 0.5\text{mm}$) on the prepared surface with needle tip, microbrush or gently air-blow. Leave for 20 sec.

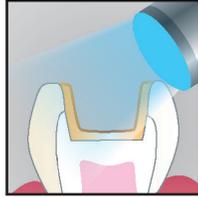

- Light cure for 5 sec (LED).

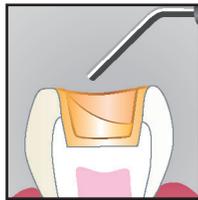


- Build additional increments ($\leq 2\text{mm}$) with universal composite such as BF II LS or other composite of choice.
- Light cure additional increments according to manufacturer's instructions.
- Finish and polish.


Restorative (F03 Viscosity)



- Clean and gently air blow the prep. Then apply FIT SA.
- Spread in a thin layer ($\leq 0.5\text{mm}$) on the prepared surface with needle tip, microbrush or gently air-blow. Leave for 20 sec.

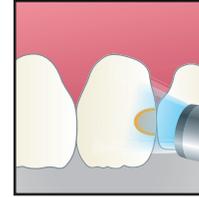

- Light cure for 5 sec (LED).

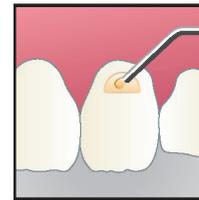

- Apply additional increments ($\leq 2\text{mm}$) of FIT SA or other composite of choice.
- Light cure each increment for 10 sec (LED).
- Finish and polish.

PROVEN QUALITY

In an effort to make dentistry better, faster, easier and more productive for the clinician, Shofu Dental once again sets the standard with the ultimate restorative material—**FIT SA**. Only Shofu has the proven expertise in bonding materials and resin restoratives necessary to make this technological leap.

CONFIDENCE

For almost 100 years, Shofu Dental's dedication to quality has made us one of the world's most respected dental products manufacturers. What can we do for you?

*Gordan V.V., et al. A clinical evaluation of a self-etching primer and a Giomer restorative material: Results at 8 years. JAM Dent. Assoc. 2007;138:621-627

shofu.com
800.827.4638
customer-service@shofu.com



BIOACTIVE.
THE BETTER CHOICE.



FIT SA

Fast & Easy to Use



Scan Here
Instructions
for use



FIT SA F10
(High Flow) 2.2g syringe

Shade	Syringe
A1	<input type="checkbox"/> Y2560
A2	<input type="checkbox"/> Y2561
A3	<input type="checkbox"/> Y2562
A4	<input type="checkbox"/> Y2563
B1	<input type="checkbox"/> Y2564



FIT SA F03
(Low Flow) 2.2g syringe

Shade	Syringe
A1	<input type="checkbox"/> Y2550
A2	<input type="checkbox"/> Y2551
A3	<input type="checkbox"/> Y2552
A4	<input type="checkbox"/> P2553
B1	<input type="checkbox"/> Y2554