Manufacturer

SHOFU INC.

11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

SHOFU DENTAL CORPORATION

1225 Stone Drive, San Marcos,
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.

10 Science Park Road, #03-12,
The Alpha, Science Park II, Singapore 117684

02:2017-04

300030422

EN

Hardie silicone polishers are for polishing hard palladium, nickel-chrome and chrome-cobalt alloys. Hardie-Black is for final adjustments and prepolishing. Hardie-Brown is for polishing.

PRECAUTIONS

- Never exceed the maximum permissible rotation speeds described in Table A.
- Mount an unmounted wheel onto the mandrel securely.
- Insert the shank completely into the chuck of the handpiece.
- Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt and exposure of shank due to repeated use.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Avoid any use at acute angles or with excessive pressure.
- Always use an unmounted wheel in clockwise motion.
- Use local dust extractor and protective equipment while using.
- Avoid exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than the intended use.

STERILIZATION

- Perform autoclave for 3 min at 134 °C (273 °F) or 30 min at 121 °C (250 °F) for sterilization.
- When using intraorally, sterilize before each use.
- For reuse after using intraorally, remove any contaminants immediately with cleaning liquid or disinfectant, then sterilize.
- Do not immerse into chlorine disinfectant for a prolonged time to avoid corrosion.
- Do not perform dry heat sterilization.
- Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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MAXIMUM PERMISSIBLE ROTATION SPEED (Table A)

Floppie and BP1	15,000 min ⁻¹
Unmounted wheel	20,000 min ⁻¹
Others	30,000 min ⁻¹

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