



ROBOT POINTS PRO-CUT

en Instructions for use

IFU-000376
01.0:2025-01

READ CAREFULLY BEFORE USE

Dental diamond bur FG

INTENDED PURPOSE

Indications for use and clinical benefit: Contouring and finishing of natural teeth, bones and dental prostheses **Intended patient population:** From child to adult

PRECAUTIONS AND WARNINGS

●Use local dust extractor, dust protective mask, etc., while using this product to avoid harmful influence of the dust on the human body. ●Use protective equipment such as glasses while using to avoid damages to eyes. ●In case any serious health hazards occur on a patient or an operator by use of this product, please report to local competent authority, our local distributor and sales company. Contact information on our sales companies is available on www.shofu.com.

DIRECTIONS FOR USE

●Insert the shank completely into the chuck of the handpiece, following the manufacturer's instructions. ●Perform test run to check safe and smooth operation without run-out prior to use. ●Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation and dirt. ●Never exceed maximum permissible rotation speeds. ●Avoid any use at acute angles or with excessive pressure. ●Use intermittently with gentle touch and water spray. ●Maximum number of reuse is 20 times. ●Do not use this product for any purposes other than specifically outlined in these instructions for use. ●This product is intended for use by dental professionals only.

REPROCESSING

<Notes> ●Perform reprocessing the instruments before each use. ●Do not use chlorine-based disinfectant or peracetic acid to avoid corrosion of the shank. ●Follow each manufacturer's instructions for use of CIDEZYME/CIDEX OPA, washer disinfectant or autoclave sterilizer. **<Initial treatment after use>** ●After using intraorally, cleaning must be started within one hour. **<Manual cleaning and disinfection>** Use CIDEZYME/CIDEX OPA for manual cleaning/disinfection respectively. After cleaning and disinfecting, thoroughly remove the cleaning agent and disinfectant with water. Wipe off moisture thoroughly with a cloth and then let it dry completely. **<Mechanical cleaning and disinfection>** Use CIDEZYME/CIDEX OPA for manual pre-wash. Clean and disinfect with washer disinfectant that complies with ISO 15883. **<Sterilization>** ●Insert into a sterilization pouch and seal securely. Then, perform autoclave at 134 °C/273 °F for 3 min. ●Use a Class B autoclave sterilizer. **<Storage>** ●After sterilization, the pouch must not be opened until next time you use to maintain its sterility. ●Store in clean and dry place. **For more information about cleaning, sterilization, and disinfection, visit our website at www.shofu.com.**

STORAGE

●Store at room temperature (1-30 °C/34-86 °F). ●Avoid exposure to moisture, corrosive chemical and its vapor. ●Keep away from direct sunlight, external pressure (physical load) and any contamination.

DISPOSAL

Dispose the contents and containers in accordance with relevant local and national regulations.

POSSIBLE SIDE EFFECTS/RESIDUAL RISKS

Should ROBOT POINTS PRO-CUT not be cleaned and sterilized according to the instructions, there is a risk of cross-infection.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

Location of the symbols glossary:https://www.shofu.com/en/symbols_glossary/



SHOFU INC.

11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0983, Japan

www.shofu.com

SHOFU DENTAL CORPORATION

1225 Stone Drive, San Marcos, California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.

1 Science Park Road, #01-01/02 The Capricorn, Singapore Science Park II, Singapore 117528