# OneGloss Mini-point

**en** Instructions for use

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**sv** Bruksanvisning

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04.02020-11  
300030339

READ CAREFULLY BEFORE USE

## Manufacturer

SHOFU INC.  
11 Kamitakamatsu-cho, Fukuine,  
Higashiyama-ku, Kyoto 605-0983, Japan  
[www.shofu.com](http://www.shofu.com)

**EC REP**

SHOFU DENTAL GmbH  
An der Pönt 70, 40885 Ratingen, Germany


SHOFU DENTAL CORPORATION  
1225 Stone Drive, San Marcos,  
California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.  
10 Science Park Road, #03-12,  
The Alpha, Science Park II, Singapore 117684

**CE** 0123

# Mini-point REF 0186

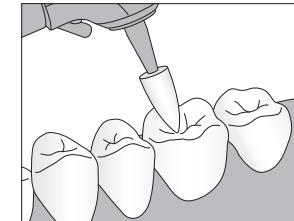Maximum permissible speed  
15,000 min<sup>-1</sup>

For  
Fissure,  
Cervical

**en**

## PRODUCT DESCRIPTION

OneGloss is a one-step finisher and polisher that provides the ideal finish for composite restorations, simply by altering the contact pressure as you go from finishing to polishing.

## INDICATIONS FOR USE

- Finishing and polishing of composite restoration
- Removal of excess resin or cement on tooth surface
- Removal of cements and polishing of enamel after removing and debonding orthodontic brackets
- Polishing of enamel after tooth scaling
- Removal of stain from tooth surface

## PRECAUTIONS AND WARNINGS

1. Do not use damaged or defective instrument, i.e. deformation, dirt, corrosion, exposure of shank due to repeated use.
2. Use protective mask and eyewear when operating handpieces.
3. Do not use this product for any purposes other than specifically outlined in these instructions for use.
4. The product is intended for use by dental professionals only.

## DIRECTIONS FOR USE

1. Insert the shank completely into the chuck of the handpiece (micro motor), following the manufacturer's instruction. Perform test run to check safety and smooth operation without run-out prior to use.
2. Apply heavy pressure for finishing and feather light pressure for polishing.

Shape	Application	Recommended pressure	
		Finishing	Polishing
Mini-point	Fissure, cervical	1.0 N	0.3 N

➢ Use at rotation speed of between 3,000 min<sup>-1</sup> to 10,000 min<sup>-1</sup> with intermittent water spray. Avoid exceeding maximum permissible speed, 15,000 min<sup>-1</sup>.

➢ Use saliva ejector or suction to eliminate moisture and debris during the procedure.

3. After using OneGloss Mini-point, remove any contaminants immediately, disinfect and sterilize.

## STORAGE AND STERILIZATION

OneGloss Mini-point should be stored away from direct sunlight, moisture and corrosive chemicals/vapor. Avoid external pressure (physical load) and contamination.

When using intraorally, sterilize before use. For reuse after using, remove any contaminants immediately with washer disinfector, then sterilize.

(Autoclave)

Shape	Cycle Type	Temperature	Holding time	Dry time
Mini-point	Pre-vacuum 3 times	134 °C 273 °F	3'00"	10'00"

Follow the instructions for use of washer disinfector and sterilizer, respectively.

Chlorine disinfector must not be used to avoid corrosion of the instruments.

For the details about REPROCESSING procedures, visit [www.shofu.de](http://www.shofu.de).

## DISPOSAL

Dispose in accordance with relevant local and national regulation.

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional**

Location of the symbols glossary: [https://www.shofu.com/en/symbols\\_glossary/](https://www.shofu.com/en/symbols_glossary/)

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