PRG Barrier Coat

EN Instructions for use

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06:2019-03
71639

Manufacturer

SHOFU INC.

11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku,
Kyoto 605-0983, Japan (www.shofu.com)

EC REP

SHOFU DENTAL GmbH

Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL CORPORATION

1225 Stone Drive, San Marcos, California 92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.

10 Science Park Road, #03-12, The Alpha, Science Park II, Singapore 117684

CE 0123

READ CAREFULLY BEFORE USE

EN LIGHT-CURED PROTECTION COATING MATERIAL VARNISH/DESENSITIZER

PRODUCT DESCRIPTION

PRG Barrier Coat is a light-cured protection coating material for coating tooth surface and desensitizer to relief dentinal hypersensitivity.

INDICATIONS FOR USE

- Coating tooth surface
 - a) Hard to brush area
 - Areas surrounding orthodontic brackets
 - Hard to brush areas due to crowded teeth
 - Areas surrounding clasps
 - b) High caries risk areas
 - Exposed root surface
 - Newly erupted molars
 - White spots
- Desensitizer
 - c) Areas of hypersensitivity

PRECAUTIONS AND WARNINGS

1. Do not use this product on patients with known allergies to methacrylate monomer.
2. Operators with known allergies to this material and/or methacrylate monomer must not use this product.
3. If any inflammation or other allergic reactions occur in either the patient or operator, immediately discontinue use and seek medical advice.
4. Use protective plastic gloves and glasses to avoid sensitization to this product. In case of accidental contact with oral soft tissue or skin, immediately blot with alcohol moistened cotton ball, and rinse with plenty of water. In case of contact with eyes, immediately flush the eyes with plenty of water and seek medical advice.
5. If this product comes in contact with oral soft tissue, the tissue may blanch, however, it should resolve in 2 or 3 days. Avoid touching or brushing the affected area.
6. Base component is for single use only.
7. Follow the instructions for use of each dental material, equipment or instrument to be used in conjunction with this product.
8. Do not use this product for any purposes other than specifically outlined in these instructions for use.
9. Use this product within the expiration date indicated on the package and container.
(Example: □ YYYY-MM-DD → Year-Month-Date of the expiration date)
10. This product is intended for use by dental professionals only.

DIRECTIONS FOR USE

1. Pretreatment
 - 1-1. For protective coating for tooth surface
Clean the tooth surface according to the standard method.
 - 1-2. For prevention of hypersensitivity
When the treating surface is contaminated with saliva or plaque, clean completely using a cotton ball etc.

After cleaning the tooth surface, rinse with water and dry thoroughly. When air stream cannot be used due to hypersensitivity, dry the area by blotting with a cotton pellet etc.

2. Snap off one of the Base containers from others along perforation.
 - When storing these materials in a refrigerator, allow them to reach room temperature before use.
3. Twist the tab while pulling up to open the container. Put one drop of Active into the container. Mix Base and Active in the Base container using disposable brush provided.
 - Use the material within 2 minutes after mixing since the mixture becomes more viscous as the reaction proceeds over time.
 - Tightly close the cap of Active immediately after each use.
4. Apply a thin layer of the mixture on the dried tooth surface and leave undisturbed for more than 3 sec.
 - Remove excess material on the disposable brush by lightly pressing it against the edge of the mouth of the Base container before application.
 - Use a new disposable brush for every patient to prevent cross infection.
 - Use the mixed material on one patient only to prevent cross contamination.
 - If the mixture applied on the tooth surface becomes contaminated with saliva, etc. prior to light-curing, remove the mixture with gauze and perform the application procedure again.
5. Light-cure using a dental light-curing unit. Repeat the light-curing procedure, if necessary, until the entire area of coating is light-cured.

Light-curing unit	Halogen	LED
Light-curing time	10 sec.	10 sec.
Wavelength	400 - 500 nm	440 - 490 nm
Light intensity	≥ 500 mW/cm ²	≥ 1000 mW/cm ²

- Use a dental light-curing unit with the above specifications.
 - Operators must use protective glasses or light shielding plate to avoid looking directly at the curing light. Operators must also protect patients' eyes from direct exposure to the curing light.
 - Disinfect the dental light-curing unit with alcohol after each use to avoid cross-contamination.
6. After light-curing, remove uncured layer by gently rubbing the surface with a water moistened cotton ball.
 7. Maintenance control after application
 - 7-1. Instruct patient to refrain from consuming any staining food and drink (e.g. curry, coffee, tea, red wine, etc.) for 3 days after coating.
 - 7-2. The coating layer of this product will be gradually worn by daily tooth brushing, and eventually disappear from the tooth surface. If significant tooth discoloration is observed in patient at follow-up visit, remove the whole coating layer using an explorer, etc.

COMPOSITION

Base: S-PRG filler based on fluoroboroaluminosilicate glass, Distilled water, Methacrylic acid monomer, and others
Active: Phosphonic acid monomer, Methacrylic acid monomer, Bis-MPEPP, Carboxylic acid monomer, TEGDMA, Polymerization initiator, and others

STORAGE

Store in refrigerator (1-10 °C/34-50 °F). Avoid high humidity and high temperature. Keep away from direct sunlight and any source of ignition.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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