

## RECOMMENDED TYPES OF STERILIZATION

INSTRUMENTS	TYPES OF STERILIZATION				
	Dry Heat Oven	Steam Autoclave	Chemical Vapor	ETO (Ethylene Oxide)	Cold Sterilization
<b>DIAMOND POINT</b>					
Shofu Robot Points	√	√	√	√	√
Hybrid Points	√	√	√	√	√
TPE Points, GingiCurettagge Points	√	√	√	√	√
<b>CARBIDE BURS</b>					
Shofu Carbide FG Finishers	√	√	√	√	√
Shofu Carbide CTF-FG Finishers	√	√	√	√	√
Shofu Carbide HP Cutters	√	√	√	√	√
<b>STONES</b>					
Dura-Green Stones, Fine & DIA	√	√	√	√	√
Dura-White Stones	√	√	√	√	√
Lab Series Stones	√	√	√	√	√
<b>SILICONE POINTS</b>					
Brownie, Greenie, Supergreenie	∅	√	√	√	√
Hardie, AcryPoint	∅	√	√	√	√
Ceramisté, Ceramisté Soft	∅	√	√	√	√
CeraMaster Coarse & CeraMaster	∅	√	√	√	√
CoreMaster Coarse & CoreMaster	∅	√	√	√	√
CompoSite, CompoSite Fine	∅	√	√	√	√
CompoMaster Coarse & CompoMaster	∅	√	√	√	√
OneGloss Mini Points	∅	√	√	√	√
ZiLMaster	∅	√	√	√	√
<b>BUR BLOCKS</b>					
HP, CA, FG, CA/FG, Original CA/FG	√	√	∅	∅	√
BurButler	∅	√	∅	∅	√
<b>DISPOSABLE FINISHING &amp; POLISHING</b>					
OneGloss PS & OneGloss	N/A	N/A	N/A	N/A	N/A
Super-Snap, Super-Snap X-Treme Disk	N/A	N/A	N/A	N/A	N/A
Super-Snap Buff Disk	N/A	N/A	N/A	N/A	N/A
Super-Snap SuperBuff Disk	N/A	N/A	N/A	N/A	N/A
Super-Snap Plastic Mandrel	N/A	N/A	N/A	N/A	N/A
<b>LEGEND</b>	√ = RECOMMENDED      ∅ = NOT RECOMMENDED      N/A = DISPOSABLE				
<b>Do NOT Use</b> ultrasonic cleaner for stones, polishers & bur blocks					

	Cleaning	Sterilization
Jet Carbides Bur	Remove the unwanted substance using brush	Dryclave 180-200 C/1 Hr Chemiclave EOG
Dura-Green, Dura-White	Remove the unwanted substance using brush	Autoclave 132 C/ 10min
Silicone Points	Remove the unwanted substance using brush	Autoclave 132 C/ 10min