Robot® Carbide CTF Kit

For Cosmetic Trimming & Finishing

English

The Robot Carbide CTF Kit contains 12 kinds of Round End Cone Burs, with 4 different lengths and 3 choices of the number of blades; 8 blades, 16 blades (F) and 30 blades (UF). These burs are used for trimming and finishing esthetic composite restorations.

The tip of each bur is designed with a safe-end to avoid damage to the gingiva.

Depending on the application, select the shape and use step by step starting with an 8-bladed pattern for trimming, followed by a 16-bladed pattern of the same shape and then using a corresponding 30-bladed pattern as the last step for finishing.

We recommend using CompoMaster or the Super-Snap Red disk for superpolishing after using the Robot Carbide CTF burs.

Details and order-information of Robot Carbide CTF (Robot Carbide CTF KIT PN0773)

	Length of working part	Recommended regions to trim and finish
CTF3	3.0 mm	Gingival and proximal margins
CTF4	4.2 mm	Curved marginal surface
CTF6	6.0 mm	Lateral incisors and premolars
CTF9	9.0 mm	All long teeth and larger surfaces on central upper incisors and canines

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SHOFU INC., 11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605 - 0983, Japan.
SHOFU INC., Singapore Branch Bldg, 203 Henderson Road (Wing B) # 09-09, Henderson Industrial Park, Singapore 159546.
SHOFU INC., Shanghai Rep. Office, Room 1109, Hongqiao Building, 421 Hongqiao Road, Shanghai 200233, China.
SHOFU DENTAL CORPORATION 4025 Bohannon Drive, Menlo Park, California 94025, USA.
SHOFU DENTAL PRODUCTS LTD. Duke's Factory, Chiddingstone Causeway, Tonbridge, Kent, TN11 8 JU, UK.
SHOFU DENTAL GmbH, Am Brühl 17, 40878 Ratingen, Germany, EU REPRESENTATIVE.