



SHOFU DIAMOND POINTS

Dental diamond burs

EN Instructions for use

CE 0120 W000003
02:2015-02

READ CAREFULLY BEFORE USE

INDICATIONS

Finishing natural teeth and restorations

PRECAUTIONS

1. This product is intended for used by dental professionals only.
2. Do not use this product for any purpose other than specifically outlined in INDICATIONS in these instructions for use.


DIRECTIONS FOR USE


1. Sterilize diamond burs before each use.
 - Sterilization method: autoclave this product for 3 min at 134 °C (273 °F) or 30 min at 121 °C (250 °F).
 - Remove any contaminants, using cleaning liquid or disinfectant and sterilize with any of the following methods; autoclave, chemiclave, heat sterilization or disinfectant liquid.
 - Do not immerse diamond burs into chlorine disinfectants.
 - Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.
2. Mount a diamond bur on a dental handpiece (dental air turbine, micro motor etc.).
 - Insert the shank completely into the chuck of the handpiece, following the manufacturer's instructions.
 - Perform test run to check safety and smooth operation without run-out prior to use.
 - Make sure that head of a diamond bur is in good condition before use. Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt, exposure of shank due to repeated use.
3. Use following the conventional manner.
 - Never exceed the maximum permissible rotating speed.
 - Use local dust extractor, protective mask, etc while finishing to avoid adverse effect of dust to the human body.
 - Use protective equipment such as glasses while using.
 - Avoid any use at acute angles or with excessive pressure to thin and long abrasives. They are fragile and may bend or fracture under excessive loads.
4. For reuse after using them intraorally, remove any contaminants immediately using cleaning liquid or disinfectant and sterilize.

PRODUCTS

1. Robot Points
2. Hybrid Points
3. T&F Hybrid Points
4. TPE Diamonds
5. GingiCurettage

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

 **Manufacturer**
SHOFU INC.
11 Kamitakamatsu-cho, Fukuine,
Higashiyama-ku, Kyoto 605-0983, Japan
www.shofu.com

 **Authorized Representative**
SHOFU DENTAL GmbH
Am Brüll 17, 40878 Ratingen, Germany

SHOFU DENTAL CORPORATION
1225 Stone Drive, San Marcos, California
92078-4059, USA