# Super-Snap/Super-Snap Polystrips

## ENGLISH

For contouring, finishing and polishing all microfilled and hybrid composites. All Super-Snap disks are manufactured without metal centers or leaving the mandrels exposed. For interproximal finishing and polishing, use Super-Snap Polystrips.

### Dura-White Stones

For fine finishing enamel, composites, compomers and porcelain, made of micrograined aluminium oxide grits.

### CompoSite Fine Polishers

For polishing micro- and hybrid composites.

### Super-Snap Details and Order Information:

	Contouring	Finishing	Polishing	Superpolishing
<b>Super-Snap Disks</b> (50 pcs./box)	 L506 L507 <b>Black</b> (coarse)	 L508 L509 L522 <b>Violet</b> (medium)	 L501 L519 <b>Green</b> (fine)	 L502 L521 <b>Red</b> (Superfine)
<b>Super-Snap Mandrel</b> (6 pcs./box)	 HP - PN 0437		 FG - PN 0438	 CA - PN 0439
<b>Super-Snap Polystrips</b> (100 pcs./box)	 L525 <b>Black/Violet</b> (coarse/medium)		 L526 <b>Green/Red</b> (fine/superfine)	

	Recommended r.p.m.	Maximum r.p.m.	Recommended pressure
Super-Snap Disks	10,000 - 12,000	15,000	0.3 - 0.6 N
Dura-White Stones	5,000 - 20,000	30,000	2N
CompoSite Fine	10,000 - 12,000	15,000	0,5 N

Super-Snap Disks and Polystrips are disposable. Dura-White Stones and CompoSite Fine are autoclavable for reuse.

### Caution

- \*Operate each abrasive up to maximum permissible speed only.
- \*Insert the shank completely into the chuck of handpiece.
- \*Rotate each abrasive each time prior to use to check smooth run and safety.
- \*Operate at recommended pressure only.
- \*Do not use at an acute angle.
- \*Wear eye protecting glasses when using.
- \*Use these products for indicated purpose only.

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SHOFU INC., 11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605, Japan.  
 SHOFU INC., Singapore Branch, 257 Selegie Road #16-279, Selegie Complex, Singapore 188350.  
 SHOFU INC. CHINA, Shanghai Office, Rm. 801-805, No. 8 Kai Cheng Business Bldg., Lane 135 Long Cao Road, Shanghai 200233, China.  
 SHOFU DENTAL CORPORATION, 4025 Bohannon Drive, Menlo Park, California 94025, USA.  
 SHOFU DENTAL PRODUCTS LTD., Duke's Factory, Chiddingstone Causeway, Tonbridge, Kent, TN11 8 JU, UK.  
 EU REPRESENTATIVE: SHOFU DENTAL GmbH, Am Brüll 17, 40878 Ratingen, Germany.