- en** Instructions for use
- fr** Mode d'emploi
- es** Instrucciones de uso
- sv** Bruksanvisning

- de** Gebrauchsanleitung
- nl** Gebruiksaanwijzing
- it** Istruzioni per l'uso
- pt** Instruções de uso

**Manufacturer**  
SHOFU INC.  
11 Kamitakamatsu-cho, Fukuine,  
Higashiyama-ku, Kyoto 605-0983, Japan  
www.shofu.com


**EC REP**  
SHOFU DENTAL GmbH  
An der Pönt 70, 40885 Ratingen, Germany

SHOFU DENTAL CORPORATION  
1225 Stone Drive, San Marcos, California  
92078-4059, USA

SHOFU DENTAL ASIA-PACIFIC PTE.LTD.  
10 Science Park Road, #03-12,  
The Alpha, Science Park II, Singapore 117684


**Super-Snap SuperBuff**

**Super-Snap Mandrel CA**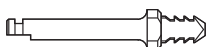


REF 0536

REF 0439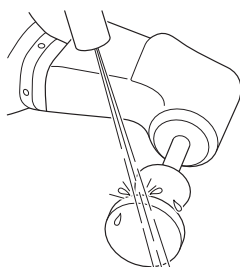

Wet the surface to be polished and moisten the SuperBuff disk with water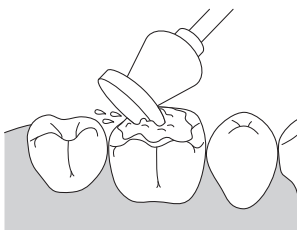


Apply uniform and feather light pressure for polishing

Max. 15,000 min<sup>-1</sup>

Do not re-use disk

**Caution**

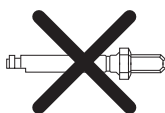Do not use OneGloss Mandrel for SuperBuff disk.

(OneGloss Mandrel CA)

**PRODUCT DESCRIPTION**

Super-Snap SuperBuff is a paste-impregnated disposable disk designed for high gloss polishing of composite restorations. Before use, slightly moisten the SuperBuff Disk with water. During use, it develops a polishing compound on the surface which leads to a high gloss in a short time.

**INDICATIONS FOR USE**

Polishing of composite restorations

**PRECAUTIONS AND WARNINGS**

1. Super-Snap SuperBuff disks are designed for single use only. Discard the disks after use in accordance with the disposal protocol of your healthcare facility.
2. Do not use damaged or defective disk or mandrel, i.e. deformation, dirt, corrosion.
3. Use protective mask and eyewear when operating handpieces.
4. Do not use this product for any purposes other than specifically outlined in these instructions for use.
5. The product is intended for use by dental professionals only.

**DIRECTIONS FOR USE**

1. Mount a SuperBuff disk on a dedicated metal mandrel, Super-Snap Mandrel.
  - Sterilize the metal mandrel before each use.
2. Insert the shank completely into the chuck of the handpiece (micro motor), following the manufacturer's instruction. Perform test run to check safety and smooth operation without run-out prior to use.
3. Wet the surface to be polished and moisten the SuperBuff disk with water to generate creamy white polishing paste.
4. Apply uniform and feather light pressure for polishing.
  - To polish occlusal surface, use the flat edge of the disk.
  - Do not use intermittent water. It will dilute polishing effect.
  - Avoid exceeding maximum permissible speed.

Super-Snap SuperBuff	15,000 min <sup>-1</sup>
----------------------	--------------------------

5. Rinse patient's mouth after use, when necessary.
6. After using dedicated metal mandrel, remove any contaminants immediately, disinfect and sterilize. Dispose the Super-Snap SuperBuff disks after each use.

**STORAGE AND STERILIZATION**

Super-Snap SuperBuff and Super-Snap Metal Mandrel should be stored away from direct sunlight, moisture and corrosive chemicals/vapor.

Sterilize Super-Snap Metal Mandrel before use.

For reuse after using, remove any contaminants immediately with washer disinfectant, then sterilize.

(Autoclave)

Product name	Cycle Type	Temperature	Holding time	Dry time
Super-Snap Metal Mandrel	Pre-vacuum 3 times	134 °C 273 °F	3'00"	10'00"

Follow the instructions for use of washer disinfectant and sterilizer, respectively. Chlorine disinfectant must not be used to avoid corrosion of the instruments. For the details about REPROCESSING procedures, visit www.shofu.de.

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional**

Location of the symbols glossary: [https://www.shofu.com/en/symbols\\_glossary/](https://www.shofu.com/en/symbols_glossary/)

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