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VINTAGE CPM Modelling Liquid

English

To achieve optimum results,
please refer to these
instructions.

Description

Product Name	State	Ingredients
VINTAGE CPM Modelling Liquid	Liquid	Propylene glycol

Characteristics

Modelling liquid for mixing VINTAGE MARGIN porcelain or Correction porcelain.

Application

For mixing with VINTAGE MARGIN porcelain or Correction porcelain instead of using distilled water.

Please note

Do not use with other products besides VINTAGE MARGIN porcelain or Correction porcelain.

Caution

- Do not use, or keep near fire, as it is flammable.
- Use in a well-ventilated place.
- Do not expose to high temperatures or direct sunlight.
- Keep it tightly closed if not in use.
- Do not use for purposes other than that described here.
- This product is for use by dental professionals only.

Important

Do not touch with hands. Never allow this liquid to contact the skin or eyes. In case of skin contact, wipe off with cotton wool soaked in alcohol and rinse immediately with running water. In case of eye contact, rinse immediately with copious amounts of water and contact a doctor.

If eruption or inflammation occurs, stop using this product and contact a doctor.

Storage Conditions

Avoid direct sunlight or fire, seal the cap tightly and keep at room temperature.

Keep this product away from persons other than dental professionals.

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