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## ENGLISH MIXING LIQUID

### INDICATIONS

Mixing liquid for mixing VINTAGE Body and Enamel porcelains. It is designed to keep moisture longer than distilled water.

### INGREDIENTS

Distilled water, 1,3-Butanediol

### NOTE ON USE

- Use VINTAGE Mixing Liquid instead of distilled water or mix with distilled water to obtain desired handling characteristics.
- If additional mixing is required to obtain desired handling, add distilled water.
- Do not use with Opaque porcelain.

### CAUTION

- Do not use or keep near fire as it is flammable.
- Use in a well-ventilated place.
- Do not expose to high temperature or direct sunlight.
- Keep lid tightly closed if not in use.
- Do not use for purpose other than that described here.
- This product is for use by dental professionals only.

### IMPORTANT

- Do not touch with hand. Pay particular attention to avoid contact with skin or eyes.
- In case of contact with skin, wipe off immediately with cotton wool soaked in alcohol and rinse with running water.
- In case of contact with eye, rinse immediately with plenty of water and seek medical advice.
- If skin eruption or inflammation occurs, stop using the product and seek medical advice.

### STORAGE

- Keep away from heat, direct sunlight or flame. Seal the cap tightly and keep at room temperature.
- This product must be kept away from all personnel except dental professionals.

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<http://www.shofu.co.jp>

Manufactured by  
SHOFU INC. 11 Komitekamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0983, Japan  
SHOFU INC. Singapore Branch 10 Science Park Road, #03-12, The Alpha, Science Park II, Singapore 117684  
SHOFU DENTAL CORPORATION 1235 Stowes Drive, San Marcos, California 92078-4059, USA  
SHOFU UK Riverside House, River Lawn Road, Tonbridge, Kent, TN9 1EP, UK  
SHOFU DENTAL TRADING (SHANGHAI) CO., LTD, No. 645 Jiye Road, Sheshan Industrial Park, Songjiang, Shanghai 201602, China