



SHOFU DENTAL CORPORATION

Call for Information 800 827 4638 www.shofu.com s A N $\,$ M A R C O S $\,$ C A L I F O R N I A

CAD/CAM FABRICATED ZIRCONIUM OXIDE FRAMEWORKS



Peter Pizzi, CDT

Achieve **pure** aesthetic metal-free restorations utilizing any Zirconium oxide framework – a growing trend.

porcelain based on feldspar, limits the potential for fracturing and chipping.

VINIAGE ZR developed specifically for Zirconium oxide frameworks.

The combination of VINIAGE ZR with Zirconia substructures offers unlimited possibilities when building aesthetic natural appearing crowns and bridges.

VINIAGE ZR offers a natural color reproduction even with very thin layers, because of the particularly fine and homogenous particle structure.

COLOR MATCH






Ultimate Styles Dental Laboratory - Akinobu Ogata

the bond to the framework and to achieve matching tooth color, even with thin layers (see layering chart below).

VINIAGE ZR gingival tissue shades offer choices for aesthetically challenging implant cases.

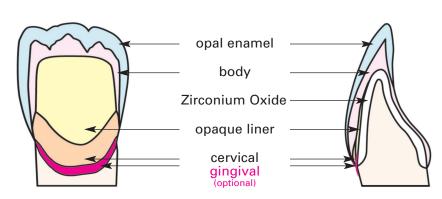




PURE opal frit - observe the clarity

NATURAL AESTHETICS

VINTAGE ZR utilizes a micro fine particle structure for the opalescent Enamel and Translucent powders, which imitate the light-optical properties of natural teeth.

VINTAGE ZR porcelain alters appearance depending on changing light conditions, to provide an expressive natural look.



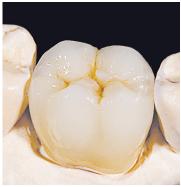





Peter Pizzi, CDT

VINIAGE ZR A UNIVERSAL ZIRCONIA PORCELAIN DESIGNED TO BE WORRY-FREE

- ZR is a pure Feldspar based ceramic, specifically developed for Zirconia frameworks
- ZR has greater strength and does not develop fractures like glass modified porcelains
- ZR is easy to handle and technician friendly

DENTAL PORCELAIN SINCE 1922.

- ZR works beautifully with any Zirconia framework
- ZR does not have graying like other Zirconia porcelains
- ZR allows for conventional cementation and has extraordinary biocompatibility
- ZR has one firing cycle for both Vintage AL and ZR metal-free porcelains, excluding the margin cycle CONCEIVED, DEVELOPED AND EXCLUSIVELY MANUFACTURED BY SHOFU, THE MAKERS OF FINE







VINIAGE ZR SYSTEM SETS

VINTAGE ZR AB SET (PN 9000)

Economic introduction to the VINTAGE ZR technique with color groups A and B

VINTAGE ZR CD SET (PN 9001)

A practical addition to the VINTAGE ZR AB set

VINTAGE ZR WHITENING SET (PN 9002)

Brighter than the "classical" tooth colors

VINTAGE ZR ENAMEL EFFECT SET (PN 9003)

Individual effects in the incisal area using opalescent and translucent Enamel Effect powders

VINTAGE ZR MARGIN PORCELAIN SET (PN 9004)

Create aesthetic margins

VINIAGE ZR COLOR EFFECT SET (PN 9005)

Natural effects to individualize Body and Cervical powders



CAD/CAN

Heat Treatme

Technical Overview

Even after several firings **IIIIIIII** ZR shows an even expansion and a secure bonding to the Zirconium oxide framework.

		Coefficient of Thermal Expansion (25 ~ 500°C)	Glass Transformation Point
Opaque Liner	2. / 4. Firing	$9.3 \times 10^{-6} K^{-1}$	620°C
Margin	2. / 4. Firing	9,3 x 10 ⁻⁶ K ⁻¹	635°C
Body	2. / 4. Firing	$9.4 \times 10^{-6} \text{K}^{-1}$	605°C
Cervical Trans.	2. / 4. Firing	$9.4 \times 10^{-6} \text{K}^{-1}$	595°C
Correction	2. / 4. Firing	9,4 x 10 ⁻⁶ K ⁻¹	585°C



MANUFACTURES

SHOFU INC. 11 Kamitakamatsu-cho, Fukuine, Higashiyama-ku, Kyoto 605-0983, Japan

SHOFU INC. Singapore Branch 10 Science Park Road #03-12, The Alpha, Science Park II, Singapore 117684
SHOFU DENTAL CORPORATION 1225 Stone Drive, San Marcos, CA 92078-4059, USA
SHOFU DENTAL PRODUCTS LTD. Duke's Factory, Chiddingstone Causeway, Tonbridge, Kent, TN11 8 JU,UK
SHOFU DENTAL SUPPLIES (Shanghai) CO. LTD. No.645 Jiye Road, Sheshan Industry Park, Songjiang 201602, Shanghai, China
EU REPRESENTATIVE: SHOFU DENTAL GmbH Am Brüll 17, 40878 Ratingen, Germany