ZiLMaster Adjustment Kit

EN Instructions for use
DE Gebrauchsanleitung
FR Mode d'emploi
NL Gebruiksaanwijzing
ES Instrucciones de uso
IT Istruzioni per l'uso

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EN READ CAREFULLY BEFORE USE

ZiLMaster Adjustment Kit is a complete system for contouring, finishing and polishing Zirconia and Lithium disilicate restorations; Dura-Green DIA grinds and finishes restorations, ZiLMaster three step system smoothens the restoration and removes surface scratches during contouring, and provides high gloss polish. ZiLMaster Adjustment Kit is available in HP and CA for both extra and intra-oral application.

CONFIGURATIONS

Dura-Green DIA: TC4, WH6, KN7	1 each
ZiLMaster Coarse: Bullet, WH6, KN7	1 each
ZiLMaster Medium: Bullet, WH6, KN7	1 each
ZiLMaster Fine: Bullet, WH6, KN7	1 each
CA type	
Dura-Green DIA: FL3, RD3	1 each
ZiLMaster Coarse: Mini-Point, Cup, Knife	1 each
ZiLMaster Medium: Mini-Point, Cup, Knife	1 each
ZiLMaster Fine: Mini-Point, Cup, Knife	1 each

INDICATIONS

Dura-Green DIA: Contouring of Zirconia or Lithium disilicate restorations
 ZiLMaster Coarse: Finishing and Pre-polishing of Zirconia or Lithium disilicate restorations
 ZiLMaster Medium: Polishing of Zirconia or Lithium disilicate restorations
 ZiLMaster Fine: Final polishing of Zirconia or Lithium disilicate restorations

PRECAUTIONS

- This product is intended for use by dental professionals only.
- Do not use this product for any purpose other than specifically outlined in INDICATIONS in these instructions for use.

DIRECTIONS FOR USE

- When using intraorally, sterilize the abrasives before each use.
 ➤ Sterilization method: autoclave for 3 min at 134 °C (273 °F) or 30 min at 121 °C (250 °F).
- Remove any contaminants, using cleaning liquid or disinfectant and sterilize with any of the following methods: autoclave, chemicleave or disinfectant liquid. Sterilize or disinfect according to each manufacturer's instructions for use.
- Do not disinfect the abrasives with chloride disinfectant liquid (sodium hypochlorite and benzalkonium chloride solution) and plasma sterilization as it may cause corrosion or the working part may come off due to degradation to bonding between the head and the shank.
- Do not perform dry heat sterilization on ZiLMaster.
- Follow the instructions for use of cleaning liquid, disinfectant and sterilizer, respectively.
- Mount the abrasive on a dental handpiece (dental electric engine, micro motor, high-speed lathe etc.).
- Insert the shank completely into the chuck of the handpiece, following the manufacturer's instructions.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Make sure that abrasive head is in good condition before use. Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt, exposure of shank due to repeated use.
- Use following the conventional manner.
- Never exceed the maximum permissible rotating speed.

Product name	Maximum permissible rotating speeds
Dura-Green DIA	30,000 min ⁻¹
ZiLMaster	20,000 min ⁻¹

- Use local dust extractor, protective mask, etc while contouring and finishing to avoid adverse effect of dust to the human body.
- Use protective equipment such as glasses while using.
- Avoid any use at acute angles or with excessive pressure to thin and long abrasives. They are fragile and may bend or fracture under excessive loads.
- Do not apply excessive pressure or exceed the recommended speed (RPM), to avoid generating frictional heat that may damage the tooth surface or pulp.
- Use SHOFU ADJUSTER to remove clogging of the debris or to adjust Dura-Green DIA to the desired size.

- For reuse after using the abrasives intraorally, remove any contaminants immediately using cleaning liquid or disinfectant and sterilize.

STORAGE

Store this product avoiding exposure to moisture, corrosive chemical and its vapor.

Keep away from external pressure (physical load) and avoid contamination.

CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional.

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