# SHOFU ZiLMaster

- en

Instructions for use

de

Gebruiksaanleitung

fr

Mode d'emploi

nl

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es

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it

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**en** READ CAREFULLY BEFORE USE

## PRODUCT DESCRIPTION

ZiLMaster is a diamond impregnated three-step polishing system designed for achieving high surface gloss on Zirconia and Lithium disilicate glass ceramic restorations; this three step system smoothens the restoration and removes surface scratches during contouring, and provides high gloss polish. ZiLMaster is available in a variety of shapes in HP and CA for both extra and intra-oral application.

## INTENDED PURPOSE

Coarse: Finishing and pre-polishing of Zirconia or Lithium disilicate restorations

Medium: Polishing of Zirconia or Lithium disilicate restorations

Fine: Final polishing of Zirconia or Lithium disilicate restorations

## PRECAUTIONS AND WARNINGS

➤Never exceed the maximum permissible rotating speed.

|                                          |                          |
|------------------------------------------|--------------------------|
| Bullet, Knife, Cup, Mini-Point, KN7, WH6 | 20,000 min <sup>-1</sup> |
| Unmounted Wheel                          | 15,000 min <sup>-1</sup> |

- Insert the shank completely into the chuck of the handpiece, following the manufacturer's instructions.
- Perform test run to check safety and smooth operation without run-out prior to use.
- Make sure that abrasive head is in good condition before use. Do not use damaged or defective instruments i.e. corrosion, surface flaw, deformation, dirt, exposure of shank due to repeated use.
- Use local dust extractor, protective mask, etc while contouring and finishing to avoid adverse effect of dust to the human body.
- Use protective equipment such as glasses while using.
- Avoid any use at acute angles or with excessive pressure to thin and long abrasives. They are fragile and may bend or fracture under excessive loads.
- This product is intended for use by dental professionals only.
- Do not use this product for any purposes other than specifically outlined in these instructions for use.

## STORAGE AND STERILIZATION

Store this product avoiding exposure to moisture, corrosive chemical and its vapor. Keep away from external pressure (physical load) and avoid contamination. When using intraorally, sterilize this product before each use. For reuse after using, remove any contaminants immediately with washer disinfector, then sterilize.

(Autoclave)

| Cycle Type            | Temperature      | Holding time | Dry time |
|-----------------------|------------------|--------------|----------|
| Pre-vacuum<br>3 times | 134 °C<br>273 °F | 3'00"        | 10'00"   |

Follow the instructions for use of washer disinfector and sterilizer, respectively. Chlorine disinfectant must not be used to avoid corrosion of the instruments. For the details about REPROCESSING procedures, visit www.shofu.de.

## DISPOSAL

Dispose in accordance with relevant local and national regulation.

**CAUTION: US Federal law restricts this device to sale by or on the order of a dental professional**

Location of the symbols glossary:  
[https://www.shofu.com/en/symbols\\_glossary/](https://www.shofu.com/en/symbols_glossary/)

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|                                          |                          |
|------------------------------------------|--------------------------|
| Bullet, Knife, Cup, Mini-Point, KN7, WH6 | 20 000 min <sup>-1</sup> |
| Unmounted Wheel                          | 15 000 min <sup>-1</sup> |

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